

# Set up of Crystal Probe System



- 1 Adjust the handle to the desired viewing position.



- 2 Connect the Surgical Probe Cable to the Control Unit. Only connect or disconnect the Cable while the unit is OFF.



- 3 Switch the Control Unit ON; after 10 seconds the system is ready.



- 4 Select the desired Nuclide setting.



- 5 Adjust Volume.



- 6 Choose the Audio Tone (located on backside of the Control Unit)



7 Set the Sample time to 1s for a one second sample time or to 10s for a ten second sample time.



8 The “counter” audio tone can be set to three different measuring ranges. Beep rate: 1x; 10x; 100x

## Calibration Check with Source Holder

Place the spot marker with the printed side down in the source holder and screw on the lid.



Put the holder with the source on the collimator sleeve until it comes to a stop.

